

Just wipe & go...

with Hygenex Disinfectant Wipes

- ✓ **One-step process**
- ✓ **Faster disinfection**
- ✓ **Great flexibility**
- ✗ **No carrying bulky bottles**
- ✗ **No aerosolisation waste**
- ✗ **No inhalation risk**




Sanitise equipment and surfaces to prevent infection – with no need to clean with detergent first.

Hygenex Disinfectant Wipes can be used to...



- **Clean and disinfect patient-shared equipment**
- **Clean non-invasive medical devices before sterilisation**
- **Sanitise surfaces where risk of infection is high**

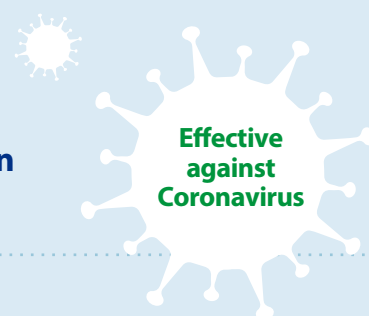

How to use:



- Step 1 -

Assess risk

Use PPE as agreed in your risk assessment guidelines.



- Step 2 -

'S-shape' technique

Remove wipe from dispenser and use a 'S-shape' technique, keeping the wipe flat (not scrunched). Move the wipe from clean to dirty. Avoid re-visiting the same place with the same wipe.



- Step 3 -

Fresh wipe

Once the wipe becomes soiled or dry, get a new wipe and continue.



- Step 4 -

Dispose

Dispose of used wipes in the appropriate bin.



- Step 5 -

Dry

Allow the surface to dry before use.

Just wipe & go...

with Hygenex Disinfectant Wipes

HYGENEX



Hygenex Disinfectant Wipes are saturated with a highly effective, broad-spectrum detergent, made up of two powerful compounds:

- **Didecyldimethylammonium chloride (DDAC)** is an antiseptic disinfectant recommended for use in hospitals, hotels and industry.
- **Triamine** tricks germs into 'seeing' a different molecule so they are less likely to build up resistance.

Made from ultra-reliable Tritex®



High strength - wipes are strong both across and along their surface.



Excellent retention - ensures wetness throughout the wipe.



Effective release - active ingredients are easily transferred to surfaces.



Low linting - avoids leaving contaminated fibres on surfaces.

Microorganism contact time

BACTERICIDAL

Effective against	Standard	Kill time
E. Coli (Esbl) Nctc 10418	EN 13727	30 seconds
E. Hirae Nctc 13383	EN 13727	30 seconds
Enterococcus Hirae	EN 16615	1 minute
	EN 14561	1 minute
P. Aeruginosa Nctc 13359	EN 13727	30 seconds
Pseudomonas Aeruginosa	EN 16615	1 minute
	EN 14561	1 minute
S. Aureus (MRSA)	EN 13727	1 minute
S. Aureus Nctc 10788	EN 13727	30 seconds
Staphylococcus Aureus	EN 16615	1 minute
	EN 14561	1 minute

VIRUCIDAL

Effective against	Standard	Kill time
Murine Norovirus		
*These organisms represent Full Virucidal efficacy according to EN 14476 2013 + A1 2015 Annex A	EN 14476	1 minute
Coronavirus		
*These organisms represent envelope virus efficacy according to EN 14476 2013 + A1 2015 Annex A	EN 14476	1 minute

FUNGICIDAL

Candida Albicans	EN 16615	1 minute
	EN 13624	1 minute
	EN 14562	2 minute

Hygenex Disinfectant Wipes are designed to save users time and effort
CONTACT US TODAY AND START EXPERIENCING THE BENEFITS FOR YOURSELF.

01202 731555

sales@hygenex.co.uk

www.hygenex.co.uk

